



How to use the Low Dose Syringe with Medicine Straw

It is recommended to draw up medication using a bottle adaptor or medicine straw. The syringe, medicine straw and bottle adaptor will contain air inside the connector or tubing. To avoid having to remove the air bubbles after the medication is drawn up it is useful to purge the syringe several times before drawing up the medication. **Use a new medicine straw each time.**


Used Straws

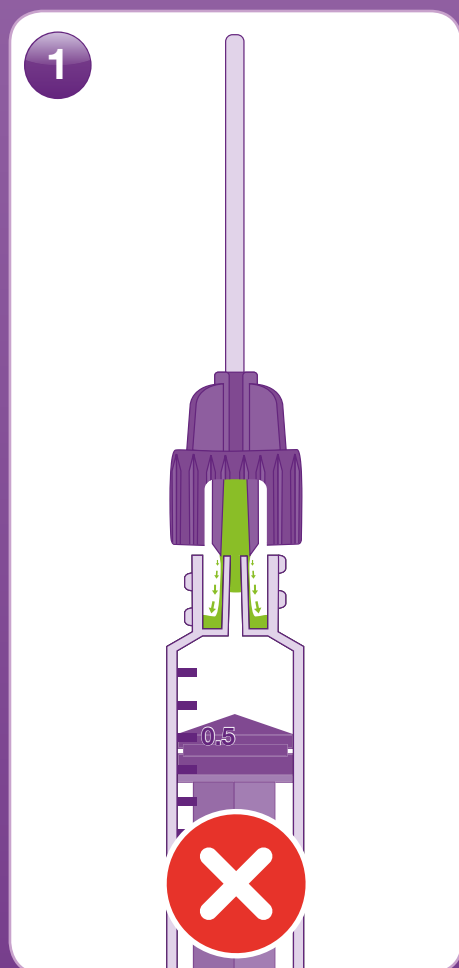


New Straws

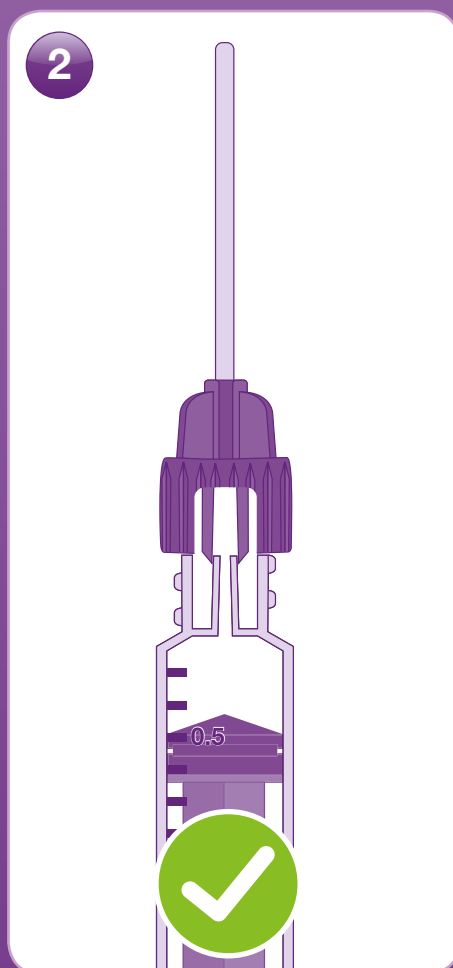

Measuring


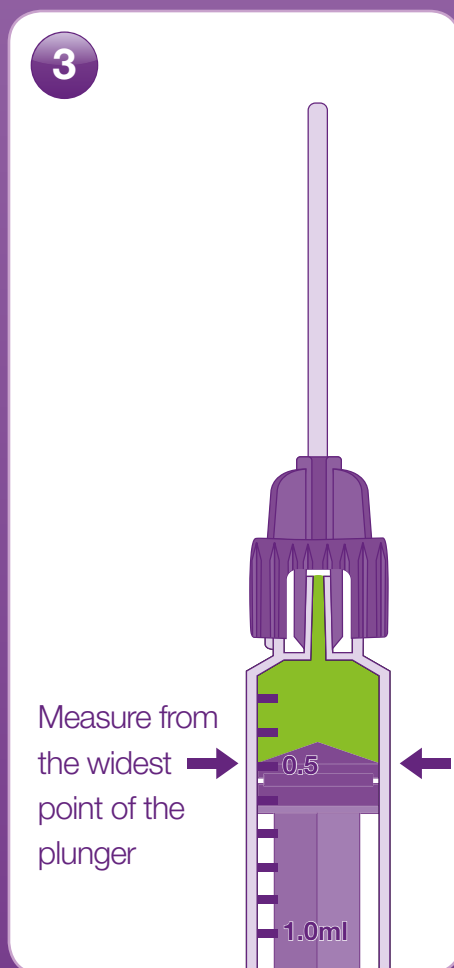


Using A Bottle Adaptor

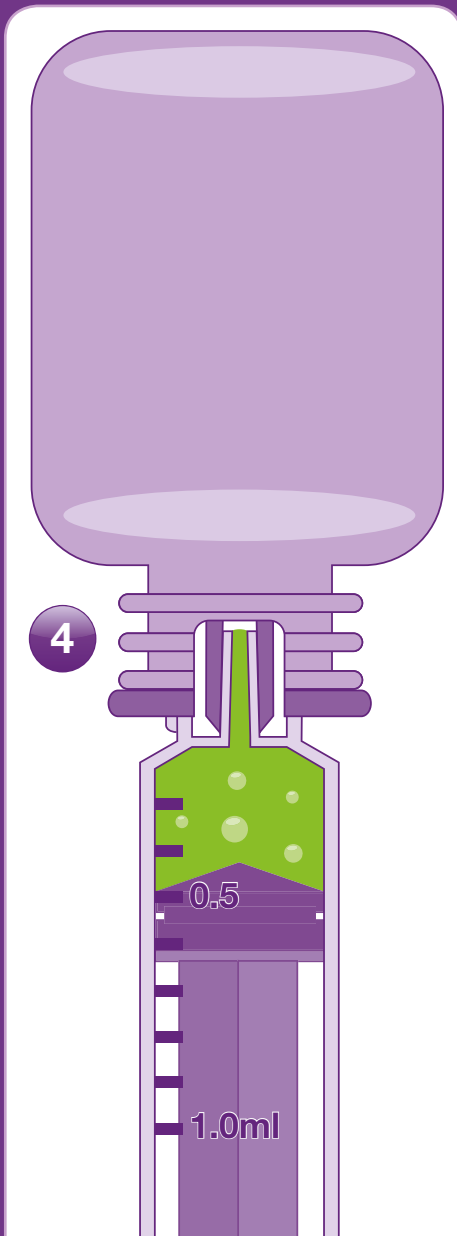

- Try to remove the air bubbles before the removal of the syringe from bottle adaptor.
- After removal of the syringe ensure the moat is clear of any excess fluid. **See illustration 8.**

Removing A Medicine Straw

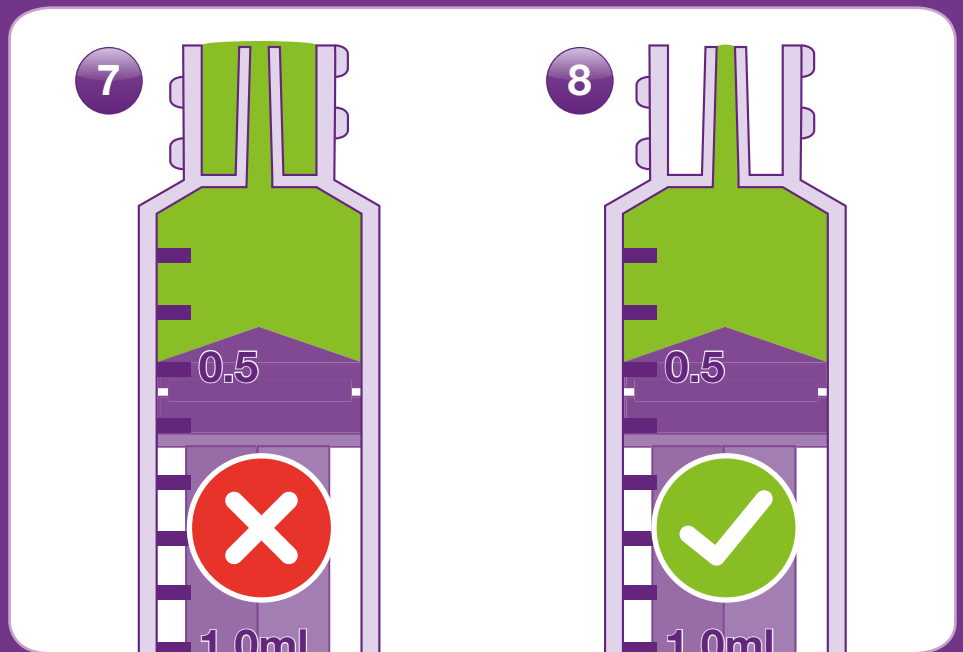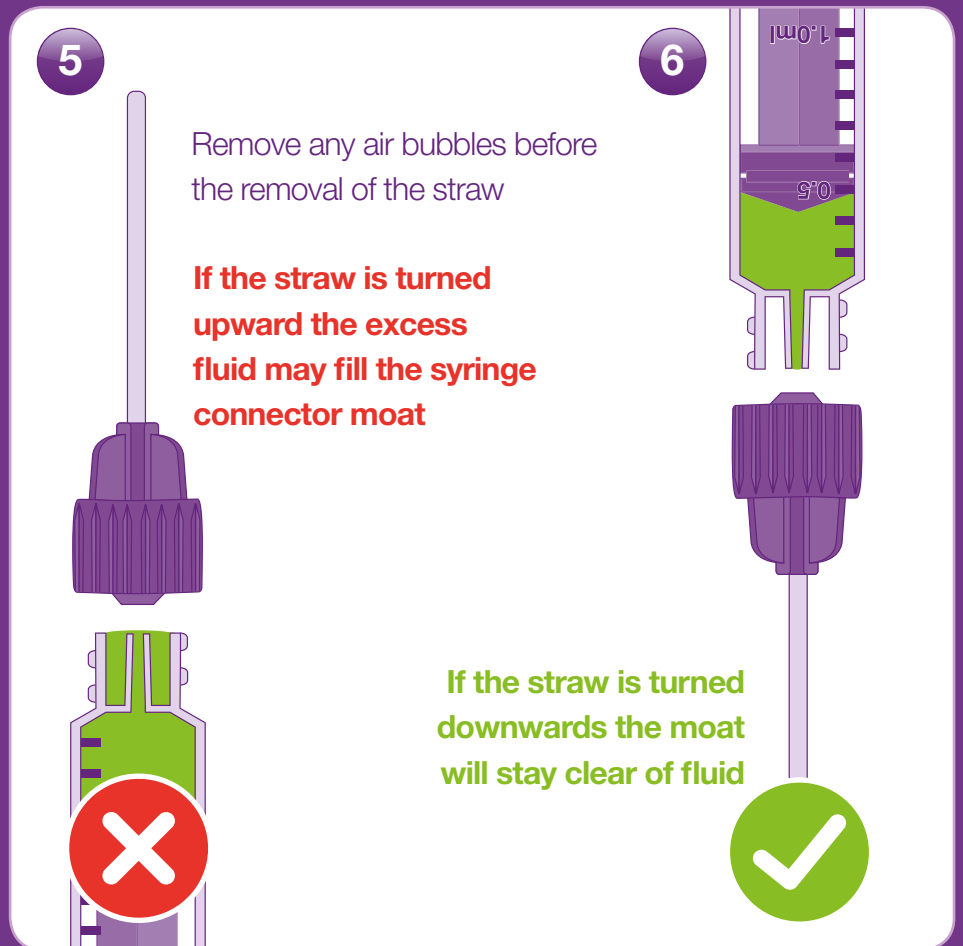

Syringe Size

In order to ensure dose accuracy medicine should be delivered using the syringe the size closest to the required dose.